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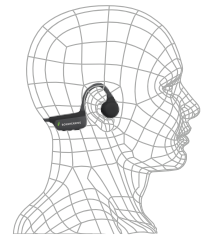
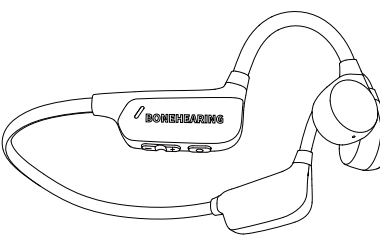
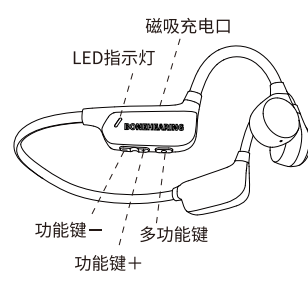
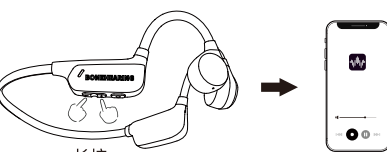
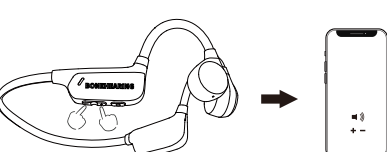
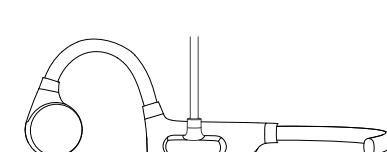
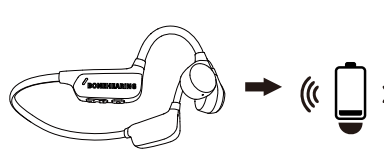
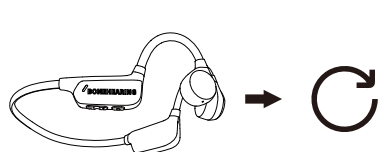
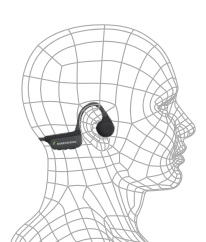
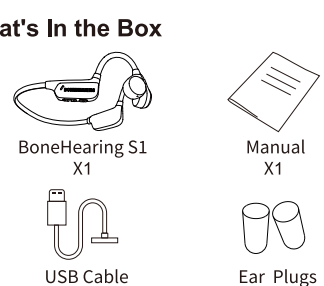
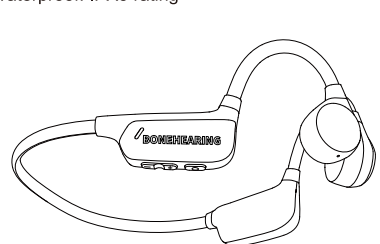
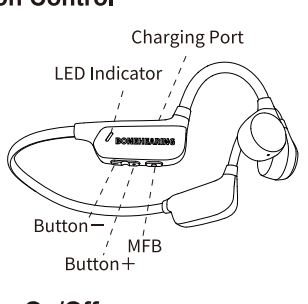
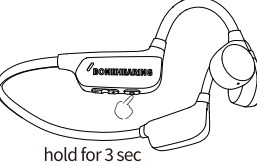
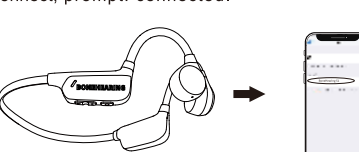
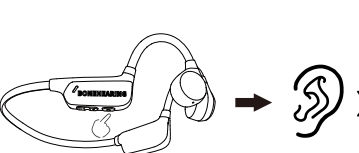
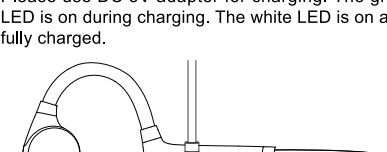
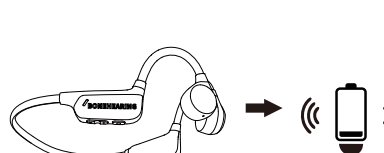
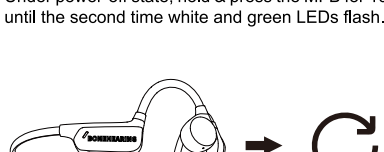
<p>Bone Conduction Sound Amplifier Headphones BoneHearing S1</p> <p>User Manual</p>	<p>How to Wear</p>  <p>What's in the Box</p> 	<p>Specifications</p> <p>Model: BoneHearing S1 Type: Bone conduction sound amplifier headphones BT Version: 5.3 Connection: ≈15m Battery: 180mAh BT Mode: ≈10h(80%volume) Amplifier Mode: ≈6h(80%volume) Charging Time: ≈2h Charging Voltage: DC 5V Waterproof: IPX5 rating</p> 	<p>Button Control</p>  <p>Power On/Off</p> <p>Power On: press & hold the MFB for 3 sec, prompt: power on/pairing Power Off: press & hold the MFB for 3 sec, prompt: power off.</p> 	<p>BT Connection</p> <p>The headset enters BT mode automatically when power-on. Select "BoneHearing S1" in BT menu to connect, prompt: connected.</p>  <p>Sound Amplifier Mode</p> <p>Double click the MFB.</p> 
<p>Answer/Hang Up/Reject A Call</p> <p>Answer/Hang Up: Click the MFB once. Reject: Double click the MFB.</p>  <p>Play/Pause Music</p> <p>Click once the MFB.</p> 	<p>Previous/Next Track</p> <p>Previous Track: Press & hold the button +. Next Track: Press & hold the button -.</p>  <p>Volume Up/Down</p> <p>Volume Up: Click the button + once. Volume Down: Click the button - once.</p> 	<p>Activate Voice Command</p> <p>Quick triple click the MFB.</p>  <p>Charge</p> <p>Please use DC 5V adapter for charging. The green LED is on during charging. The white LED is on after fully charged.</p> 	<p>Low Battery</p> <p>When battery is lower than 10%, prompt: battery low.</p>  <p>Restore Factory Setting</p> <p>Under power-off state, hold & press the MFB for 10 sec until the second time white and green LEDs flash.</p> 	<p>Attentions</p> <ol style="list-style-type: none"> Listening to loud music for a long time may cause discomfort, please use medium volume if possible. Don't use high volume for long periods of time. Using headphones will affect your perception of the surrounding environment. Pay more attention when using headphones. Do not remove or modify the headphone as this may cause damage or burn out. Do not throw, test on headphones or put them into a fire. Keep it away from fire, hydration and high pressure equipment. Not suitable for children under 10-year-old containing small parts. Do not classify it as household waste. Please send it to the local electrical equipment recycling station. <p>Note: The company does not assume any responsibility for the loss of personal property caused by the violation of the above warnings.</p>